

INSTRUCTIONS FOR USE

Garment Application



1. Adjust the back straps and open the front straps. Insert arms into the garment as done when putting on a jacket. Close using the hook and eye clip.
2. The front strap may be added, adjusted or removed as desired.

Dressing Application



1. Apply the dressing under the garment in the affected armpit as directed in the dressing instructions for use.

Garment Removal



1. Remove and discard the dressing as directed in the dressing instructions for use.
2. Open the garment using the hook and eye clip at the front of the garment.
3. Remove the garment as done when removing a jacket.

Indications for use

The garment is indicated for the retention of a dressing in the axilla area.

Intended user

Intended for home use.

Sizing

Select your size as per sizing instructions.

Guidelines

1. Garment may be worn both throughout the day and/or overnight.
2. Check the garment for damage prior to use.

Precautions

1. Ensure garment is clean before use. Soiled garments should not be used.
2. Wash garment as required. See care instructions.
3. Change dressing as directed in the dressing instructions for use or as recommended by your doctor or nurse.

Warnings

1. The garment is not a wound dressing – wound(s) should be fully covered by dressing(s).
2. If any unexpected discomfort or wound irritation occurs, please consult your doctor or nurse.
3. Do not use, if you have a known allergy or sensitivity to materials listed.
4. Device is not indicated for wounds located on or around the breast, on the chest or on the back.

Potential side effects

Dermatitis, infection and skin irritation or skin abrasion.

Care instructions

1. Close hook and eye clip before washing.
2. Wash with similar colours in a washing bag on a delicate cycle.
3. Use of chlorine based disinfection will decrease garment durability and function.






SIZING CHART

Please note

If in between sizes, please choose to suit your fit preference.

		Small		Medium		Large		X-large		XX-large	
UK		8	10	12	14	16	18	20	22	24	26
EU		36	38	40	42	44	46	48	50	52	54
USA		6	8	10	12	14	16	18	20	22	24
Bust	cm	81	91	91	101	101	111	111	121	121	131
	inch	32	36	36	40	40	44	44	48	48	52
Waist	cm	61	71	71	81	81	91	91	101	101	111
	inch	24	28	28	32	32	36	36	40	40	44

SIZING INSTRUCTIONS

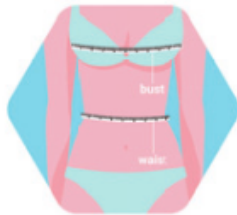

Measure around the fullest part of your bust AND measure your waist.

Material composition

78% Micro Modal, 22% Lycra® Elastane (Exclusive of trim) Trim composed of Polyester, Polyamide, Nylon coated wire.



Medical Device



Single Patient - multiple use



Caution



Keep dry



HidraMed Solutions, Business Innovation Centre, NUI Galway, Galway, H91 V207, Ireland.