

INSTRUCTIONS FOR USE

Garment Application



1. The care tag on the device indicates the reverse of the garment.
2. The garment is worn under regular clothing or in place of a T-shirt.
3. Don the garment as you would a T-shirt.

Dressing Application



1. Insert the dressing(s) inside the garment and place in the desired area of the axillary region.
2. Use external fastener as directed in the dressing instructions for use, ensuring that you tuck in the tabs of your fastener into the garment holes.

Dressing / Garment Removal

1. Remove and discard the dressing as directed in the dressing instructions for use.
2. Remove the garment as done when removing a T-shirt.

Indications for use

The garment is indicated for the retention of a dressing in the axillary region.

Intended user

Intended for home use.

Sizing

Select your size as per sizing instructions.

Guidelines

1. Garment may be worn both throughout the day and/or overnight.
2. Check the garment for damage prior to use.

Precautions

1. Ensure garment is clean before use. Soiled garments should not be used.
2. Wash garment as required. See care instructions.
3. Change dressing as directed in the dressing instructions for use or as recommended by your doctor or nurse.

Warnings

1. The garment is not a wound dressing – wound(s) should be fully covered by dressing(s).
2. If any unexpected discomfort or wound irritation occurs, please consult your doctor or nurse.
3. Do not use, if you have a known allergy or sensitivity to materials listed.
4. Device is not indicated for wounds located on the chest or back.

Potential side effects

Dermatitis, infection and skin irritation or skin abrasion.

Care instructions

1. Wash with similar colours in a washing bag on a delicate cycle.
2. Use of chlorine based disinfection will decrease garment durability and function.





SIZING CHART

Please note

If in between sizes, please choose to suit your fit preference.

		Small		Medium		Large		X-large		XX-large	
Chest	inch	34	38	38	42	42	46	46	50	50	56
	cm	86	96	96	106	106	116	116	126	126	143
Waist	inch	30	34	34	38	38	42	42	46	46	52
	cm	76	86	86	96	96	106	106	116	116	132

Material composition

89% Micro Modal, 11% Elastane
(Exclusive of trim) Trim composed of Polyurethane, Polyester Polyamide.

SIZING INSTRUCTIONS

1. Measure around your chest & waist.
2. Select size from above sizing chart.



Hidramed Solutions
Business Innovation Centre,
NUI Galway,
Galway,
H91 V207
Ireland