

INSTRUCTIONS FOR USE

Garment Application



1. The care tag on the device indicates the reverse of the garment.
2. Briefs are worn underneath or in place of regular underwear.
3. Don the briefs as you would with standard Briefs.

Dressing Application



1. Insert the dressing(s) inside the garment and place in the desired area of the groin / buttocks region.
2. Use external fastener as directed in the dressing instructions for use, ensuring that you tuck in the tabs of your fastener into the garment holes.

Dressing / Garment Removal

1. Remove and discard the dressing as directed in the dressing instructions for use.
2. Remove the garment as done when removing regular briefs.

Indications for use

The garment is indicated for the retention of a dressing in the buttocks and groin.

Intended user

Intended for home use.

Sizing

Select your size as per sizing instructions.

Guidelines

1. Garment may be worn both throughout the day and/or overnight.
2. Check the garment for damage prior to use.

Precautions

1. Ensure garment is clean before use. Soiled garments should not be used.
2. Wash garment as required. See care instructions.
3. Change dressing as directed in the dressing instructions for use or as recommended by your doctor or nurse.

Warnings

1. The garment is not a wound dressing – wound(s) should be fully covered by dressing(s).
2. If any unexpected discomfort or wound irritation occurs, please consult your doctor or nurse.
3. Do not use, if you have a known allergy or sensitivity to materials listed.
4. Device is not indicated for wounds located in areas other than the buttocks and groin region.

Potential side effects

Dermatitis, infection and skin irritation or skin abrasion.

Care instructions

1. Wash with similar colours in a washing bag on a delicate cycle.
2. Use of chlorine based disinfection will decrease garment durability and function.





SIZING CHART

Please note

If in between sizes, please choose to suit your fit preference.

		Small		Medium		Large		X-large		XX-large	
	UK	8	12	12	16	16	20	20	24	24	28
	US	6	10	10	14	14	18	18	22	22	26
	EU	36	40	40	44	44	48	48	52	52	56
Waist	inch	24	28	28	32	32	36	36	40	40	44
	cm	62	72	72	82	82	92	92	102	102	112

Material composition

78% Micro Modal, 22% Elastane
(Exclusive of Trim) Trim composed of
Polyester, Polyamide, Elastane,
Polyurethane, Transparent Rubber Print

SIZING INSTRUCTIONS

Select size from above
sizing chart.



Single Patient
- multiple use



Caution



Keep dry



Hidramed Solutions
Business Innovation Centre,
NUI Galway,
Galway, H91 V207
Ireland