




Redefining Patient led Wound Care



Hidradenitis Suppurativa Specific Wound Dressing System

HidraWear is the worlds first **HS specific** wound care product designed to make managing HS drainage quick, easy, and completely adhesive free.


It is a unique **Wearable Wound Care** system for the management of wounds in difficult to dress areas, such as the under-arm region, or buttocks, groin & upper thigh. This **NEW** dressing retention system represents a brand new category in wound care. It combines a reusable garment with a super absorbent dressing, ensuring **adhesive free dressing retention** through our patented Stick-tite technology.

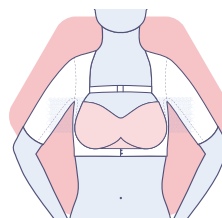


Women's Crop Top



Unisex T-Shirt

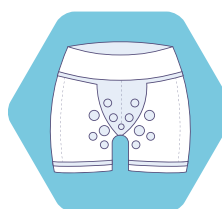

Men's Boxer Briefs

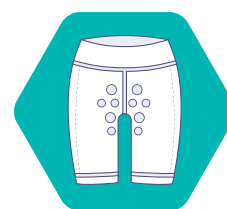

Women's Briefs


The Garments

Each garment utilizes our patented **HidraHex** technology to enable patients to easily change their dressing day or night.

Super Absorbant Dressing

3" x 5" Super Absorbent dressings lock away moisture from the wound while protecting clothing and bedlinen from exudate strike through.

Stick-tite Hook & Loop retention technology secures the dressing in place without use of any adhesives.

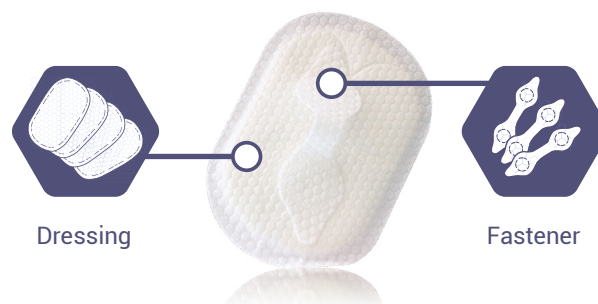

Each HidraWear wound dressing is co-packed with a patented fastener, ensuring adhesive free wound dressing retention

HidraWear Garments Incorporate:



HidraHex

The hexagon pattern is designed to minimize leaks and strike through while remaining fully breathable & promoting wicking.



Easy On

HidraWear Garments are specifically designed to be easy to get on and off, with the women's underarm crop top incorporating a front closing clasp.



SeamSense

HidraWear garments have outward facing seams. This ensures soft fabric is against your delicate skin and reduces irritation. On the shorts and briefs the seams are on the reverse and provide smooth and frictionless movement.



AeroSilk

All HidraWear garments are made of a premium micro modal and elastane blend fabric that is soft and breathable.



TrueFit

Adjustable straps in the women's underarm crop top enable a tailored fit.

HidraWear can now be prescribed to your patients, with their monthly supply delivered directly to their home

Simply fill in the order form on page 3 & email or fax along with your patient info page (See right for filled in example)

How It Works:

ORDER IS PLACED

INSURANCE IS VERIFIED

SUPPLIES ARE SHIPPED

WE PROVIDE PATIENT & CLINIC WITH STATUS UPDATES



Delivered to patient

For further support please contact us on

Call: (888) 711 2014

Email: hidrawear@rotech.com

Clinic Name ABC Dermatology Clinic Phone 123-456-7890 Clinic Fax 123-456-7890		Rotech WoundCare Complete		HidraWear																																																																									
Patient Name Jane Doe		FAX # (866) 220-8963 Fax or email* with Patient Information Sheet orders@rotech.com Phone # (888) 711-2014																																																																											
Order Date: 11-2-2022		Have patient's wound/s ever been debrided? <input checked="" type="radio"/> YES <input type="radio"/> NO		Is patient currently seen by Home Health? <input type="radio"/> YES <input checked="" type="radio"/> NO																																																																									
ICD-10 L73.2-Hidradentis suppurativa L98.499-Non-pressure chronic ulcer of skin L98.419-Non-pressure chronic ulcer of skin / Buttocks																																																																													
WOUND INFORMATION		<table border="1"> <thead> <tr> <th>ICD-10 (See ICD-10 legend above)</th> <th>Wound Thickness (HidraWear dressing requires Full)</th> <th>LOCATION</th> <th>Wound Size (cm) (L x W x D)</th> <th colspan="4">Drainage (Exudate) HidraWear Dressing requires M - H</th> </tr> </thead> <tbody> <tr> <td>Wound 1: L73.2 + 98.499</td> <td>Partial <input checked="" type="radio"/> Full</td> <td>Right Ankle</td> <td>4.5 x 1.5 x 0.5</td> <td>N</td> <td>L</td> <td>M</td> <td>H <input checked="" type="radio"/></td> </tr> <tr> <td>Wound 2: L73.2 + 98.419</td> <td>Partial <input checked="" type="radio"/> Full</td> <td>Left Buttocks</td> <td>4.6 x 2.8 x 0.2</td> <td>N</td> <td>L</td> <td>M</td> <td>H <input checked="" type="radio"/></td> </tr> <tr> <td>Wound 3:</td> <td>Partial Full</td> <td></td> <td></td> <td>N</td> <td>L</td> <td>M</td> <td>H</td> </tr> <tr> <td>Wound 4:</td> <td>Partial Full</td> <td></td> <td></td> <td>N</td> <td>L</td> <td>M</td> <td>H</td> </tr> <tr> <td>Wound 5:</td> <td>Partial Full</td> <td></td> <td></td> <td>N</td> <td>L</td> <td>M</td> <td>H</td> </tr> <tr> <td>Wound 6:</td> <td>Partial Full</td> <td></td> <td></td> <td>N</td> <td>L</td> <td>M</td> <td>H</td> </tr> <tr> <td>Wound 7:</td> <td>Partial Full</td> <td></td> <td></td> <td>N</td> <td>L</td> <td>M</td> <td>H</td> </tr> <tr> <td>Wound 8:</td> <td>Partial Full</td> <td></td> <td></td> <td>N</td> <td>L</td> <td>M</td> <td>H</td> </tr> </tbody> </table>				ICD-10 (See ICD-10 legend above)	Wound Thickness (HidraWear dressing requires Full)	LOCATION	Wound Size (cm) (L x W x D)	Drainage (Exudate) HidraWear Dressing requires M - H				Wound 1: L73.2 + 98.499	Partial <input checked="" type="radio"/> Full	Right Ankle	4.5 x 1.5 x 0.5	N	L	M	H <input checked="" type="radio"/>	Wound 2: L73.2 + 98.419	Partial <input checked="" type="radio"/> Full	Left Buttocks	4.6 x 2.8 x 0.2	N	L	M	H <input checked="" type="radio"/>	Wound 3:	Partial Full			N	L	M	H	Wound 4:	Partial Full			N	L	M	H	Wound 5:	Partial Full			N	L	M	H	Wound 6:	Partial Full			N	L	M	H	Wound 7:	Partial Full			N	L	M	H	Wound 8:	Partial Full			N	L	M	H
ICD-10 (See ICD-10 legend above)	Wound Thickness (HidraWear dressing requires Full)	LOCATION	Wound Size (cm) (L x W x D)	Drainage (Exudate) HidraWear Dressing requires M - H																																																																									
Wound 1: L73.2 + 98.499	Partial <input checked="" type="radio"/> Full	Right Ankle	4.5 x 1.5 x 0.5	N	L	M	H <input checked="" type="radio"/>																																																																						
Wound 2: L73.2 + 98.419	Partial <input checked="" type="radio"/> Full	Left Buttocks	4.6 x 2.8 x 0.2	N	L	M	H <input checked="" type="radio"/>																																																																						
Wound 3:	Partial Full			N	L	M	H																																																																						
Wound 4:	Partial Full			N	L	M	H																																																																						
Wound 5:	Partial Full			N	L	M	H																																																																						
Wound 6:	Partial Full			N	L	M	H																																																																						
Wound 7:	Partial Full			N	L	M	H																																																																						
Wound 8:	Partial Full			N	L	M	H																																																																						
Women's Crop Top 		Unisex T-Shirt 																																																																											
Women's Briefs 		Men's Briefs 																																																																											
Wound Care products required (Please Check Mark Size Requested)		Drainage Required		Wound Number (✓ for supplies)																																																																									
Hidrawear Garment (Tick/Circle style and size on sizing chart above)		-		**																																																																									
Hidrawear Dressing Pad Super Absorbent (5x3) (recommended with garment)		Mod/Heavy		30																																																																									
Carboxiflex: (4x4) (3x6) (6x8)		Mod/Heavy		30																																																																									
Xeroform: (2x2) (4x4) (5x9)		-		30																																																																									
Telfa Non-Adherent Pad: (2x2) (4x4) (4x5) (4x8) Other		-		30																																																																									
Gelling Fiber Super Absorbent: (2x2) (4x4) (6x6) (8x8)		Mod/Heavy		30																																																																									
Practitioner Name Dr. Sam Johnson, MD NPI# 987654321		Practitioner Name NPI#		Practitioner Name NPI#																																																																									
Practitioner Name NPI#		Practitioner Name NPI#		Practitioner Name NPI#																																																																									
Practitioner Name NPI#		Practitioner Name NPI#		Practitioner Name NPI#																																																																									
Signature Patient Signature*: [Signature] Date: 11-2-2022		Treating Practitioner MUST check mark their respective box																																																																											
I attest by my signature that it is my intention for the prescription to remain valid until the diagnosis described is resolved or otherwise directed by the signer. The requested supplies are medically necessary and the wound(s) has/have been debrided and/or surgically created or modified. I have instructed the patient on how to use the supplies being requested.		Duration of Treatment will be 90 Days unless specified:																																																																											
X Practitioner Signature: [Signature] Date: 11-2-2022		Signed Date: 11-2-2022																																																																											



Size Matters:

For best user experience please ensure you take your measurements prior to selecting a size. If in between sizes, downsizing is typically recommended. If you are unsure of your size, please take the time to contact our customer care team (call toll-free (888) 711 2014 or email hidrawear@rotech.com)

Rotech WoundCare Complete is an authorised distributor for HidraWear

Clinic Name	
Clinic Phone	Clinic Fax
Patient Name	

Order Date:


Have patient's wound/s ever been debrided?	YES	NO
--	-----	----

FAX # (866) 220-8963	
Fax or email* with Patient Information Sheet orders@rotech.com Phone # (888) 711-2014	
AE Name	AE ID #

Is patient currently seen by Home Health?	YES	NO
---	-----	----

ICD-10	L73.2-Hidradentis suppurativa	L98.499-Non-pressure chronic ulcer of skin	L98.419-Non-pressure chronic ulcer of skin / Buttocks
--------	-------------------------------	--	---

WOUND INFORMATION	ICD-10 (See ICD-10 legend above)	Wound Thickness (HidraWear dressing requires Full)		LOCATION	Wound Size (cm) (L x W x D)	Drainage (Exudate) HidraWear Dressing requires M - H			
	Wound 1:	Partial	Full			N	L	M	H
	Wound 2:	Partial	Full			N	L	M	H
	Wound 3:	Partial	Full			N	L	M	H
	Wound 4:	Partial	Full			N	L	M	H
	Wound 5:	Partial	Full			N	L	M	H
	Wound 6:	Partial	Full			N	L	M	H
	Wound 7:	Partial	Full			N	L	M	H
	Wound 8:	Partial	Full			N	L	M	H

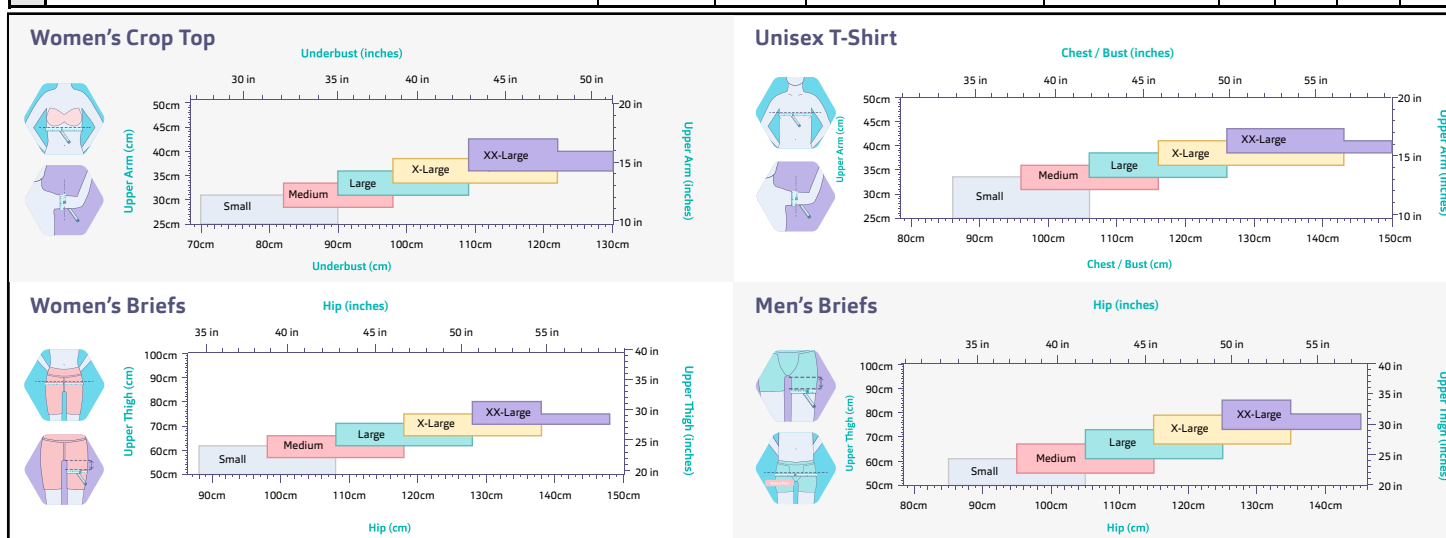

DRESSING SUPPLIES	Wound Care products required (Please Check Mark Size Requested)	Drainage Required	Max Units per Wound per Month	Frequency of Change Daily unless Spec.	Wound Number (✓ for supplies)							
					Wound #1	Wound #2	Wound #3	Wound #4	Wound #5	Wound #6	Wound #7	Wound #8
	HidraWear Garment (Tick/Circle style and size on sizing chart above)	-	**									
	HidraWear Dressing Pad Super Absorbent (5x3) (recommended with garment)	Mod/Heavy	30									
	Carboflex: (4x4) (3x6) (6x8)	Mod/Heavy	30									
	Xeroform: (2x2) (4x4) (5x9)	-	30									
	Telfa Non-Adherent Pad: (2x2) (4x4) (4x5) (4x8) Other_____	-	30									
	Gelling Fiber Super Absorbent: (2x2) (4x4) (6x6) (8x8)	Mod/Heavy	30									

✓	Practitioner Name	Practitioner Name	Practitioner Name	Practitioner Name	Practitioner Name	Practitioner Name	Treating Practitioner MUST check mark their respective box
	NPI#	NPI#	NPI#	NPI#	NPI#	NPI#	

SIGNATURE	* I request that payment of my insurance benefits for any supplies be made to Rotech Healthcare Inc. I am responsible for any balance due that is not covered by my insurance. I understand any product received in my home cannot be returned if opened. I authorize any holder of my medical information to release to Rotech Healthcare Inc. any information needed to determine benefits payable for these supplies or services. I acknowledge receipt and understanding of the patient rights that Rotech Healthcare Inc. provides to all patients.					
	Patient Signature*:					Date:
	I attest by my signature that it is my intention for the prescription to remain valid until the diagnosis described is resolved or otherwise directed by the signer. The requested supplies are medically necessary and the wound(s) has/have been debrided and/or surgically created or modified. I have instructed the patient on how to use the supplies being requested.					Duration of Treatment will be 90 Days unless specified:
	X Practitioner Signature:					Signed Date:

* If emailing, please use encrypted/secure email. Communications via unencrypted email are not secure and there is a possibility PHI may be misdirected or intercepted.



Frequently Asked Questions

What is needed to process an order?

The following information is a general checklist of the items required on a standard written order (Rotech form or any written order) by most insurance plans, including Medicare.



1

Qualified Wound Is Present

Medicare covers surgical/wound dressings when a qualifying wound is present. CMS defines a qualified wounds as either of the following:

- A wound caused by, or treated by a surgical procedure
- After debridement of the wound, regardless of the debridement technique

Document debridement TYPE used to remove devitalized or necrotic tissue from wound (examples given are not all-inclusive):

Surgical: sharp instruments or laser

Mechanical: irrigation or wet-to-dry

Chemical: topical application of enzymatic agents

Autolytic: application of an occlusive dressing to open wound

2

Wound Information

- Type of wound(s) or ICD-10 Codes
- Location of each wound
- Size of wound(s) in cm (L x W x D)
- Amount of exudate

3

Treatment Plan

- Type of dressing to use
- Size of the dressings
- Amount to be used at one time
- Frequency of the dressing change
- Expected duration (up to 90 days)

4

Provider's Information

- Provider's name and NPI
- If unable to obtain signature when order is placed, please note where signature request should be sent (if it is different than clinic)

5

Patient's Demographic Info / Face Sheet

Having this information right away allows us to begin verifying insurance benefits and helps avoid any delays in shipment.

How easy is it to order and have supplies shipped to patients?

- Rotech's one-page order form takes just 45 seconds to complete. (...yes we timed it!)
- As an added bonus, Rotech is an option when ordering via iHeal, Wound Expert & Intellicure.

How quickly are supplies delivered?

- With the exception of a PO Box address, orders are typically shipped and delivered within 24-48 hours

What if Rotech is not contracted with a patient's insurance plan?

- In most cases, Insurance is verified within an hour of receiving the order, and once approved, supplies ship the same day.
- If Rotech is not contracted with a patient's plan, we will do our very best to find a company that is, notifying you along the way!

Will Rotech substitute products when I request brand specific products?

- If there is a reimbursement issue or a product is not available, our team will confirm any changes with you prior to shipping supplies.

Our staff does not have time to deal with unnecessary phone calls. Is this something we should expect?

- Not only does our team value your time but we respect it. You are unlikely to receive any calls from us unless we are returning a call to you.
- Our team will fax order updates to confirm shipment and make you aware of any delays.