INSTRUCTIONS FOR USE

Garment Application

1. Adjust the back straps and open the front straps. Insert arms into the garment as done when putting on a jacket. Close using the hook and eye clip.
2. The front strap may be added, adjusted or removed as desired.

Dressing Application

1. Insert the dressing(s) inside the garment and place in the desired area of the axillary region.
2. Use external fastener as directed in the dressing instructions for use, ensuring that you tuck in the tabs of your fastener into the garment holes.

Garment Removal

1. Remove and discard the dressing as directed in the dressing instructions for use.
2. Open the garment using the hook and eye clip at the front of the garment.
3. Remove the garment as done when removing a jacket.

Indications for use

The garment is indicated for the retention of a dressing in the axilla area.

Intended user

Intended for home use.

Sizing

Select your size as per sizing instructions.

Guidelines

1. Garment may be worn both throughout the day and/or overnight.
2. Check the garment for damage prior to use.

Precautions

1. Ensure garment is clean before use. Soiled garments should not be used.
2. Wash garment as required. See care instructions.
3. Change dressing as directed in the dressing instructions for use or as recommended by your doctor or nurse.

Warnings

1. The garment is not a wound dressing – wound(s) should be fully covered by dressing(s).
2. If any unexpected discomfort or wound irritation occurs, please consult your doctor or nurse.
3. Do not use, if you have a known allergy or sensitivity to materials listed.
4. Device is not indicated for wounds located on or around the breast, on the chest or on the back.

Potential side effects

Dermatitis, infection and skin irritation or skin abrasion.

Care instructions

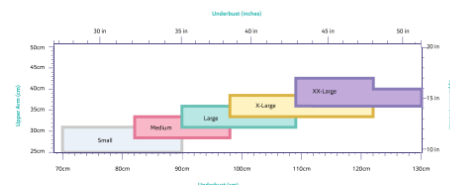
1. Close hook and eye clip before washing.
2. Wash with similar colours in a washing bag on a delicate cycle.
3. Use of chlorine based disinfection will decrease garment durability and function.

SIZING CHART

Centimetres	Small	Medium	Large	X-Large	XX-Large
Under-Bust (cm)	70	82	90	98	109
Upper Arm (cm)	26 1/2	28 1/2	31	33 1/2	36
Inches	Small	Medium	Large	X-Large	XX-Large
Under-Bust (in)	28	32 1/2	35 1/2	39	43
Upper Arm (in)	10 1/2	11 1/4	12 1/4	13 1/4	14 1/4

If an individual measurement falls between sizes, select smaller size. If Underbust and Upper Arm sizes differ, select the smaller size. See below chart for additional guidance.

SIZING INSTRUCTIONS

Underbust

Upper Arm (armpit side of the bicep)

Do not use if packaging is damaged

Dispose with appropriate care

Medical Device

Single Patient - multiple use

Patient Information Website
www.hidrawear.com

Caution

Store in dry conditions

Consult instructions for use

Manufactured by:
HidraMed Solutions,
Business Innovation Centre,
NUI Galway, Galway H91 V207,
Ireland

UK REP

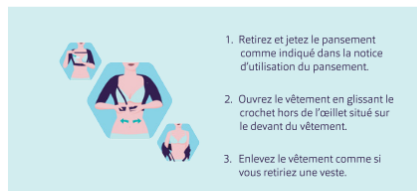
BS Partnership Ltd
41 Glen Y-Cood Road
Cardiff CF23 6NB, UK

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