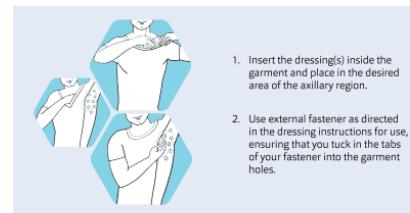
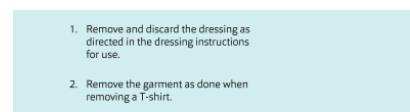
**INSTRUCTIONS FOR USE**
**Garment Application**

**Dressing Application**

**Dressing / Garment Removal**

**Indications for use**

The garment is indicated for the retention of a dressing in the axillary region.

**Intended user**

Intended for home use.

**Sizing**

Select your size as per sizing instructions.

**Guidelines**

- Garment may be worn both throughout the day and/or overnight.
- Check the garment for damage prior to use.

**Precautions**

- Ensure garment is clean before use. Soiled garments should not be used.
- Wash garment as required. See care instructions.
- Change dressing as directed in the dressing instructions for use or as recommended by your doctor or nurse.

**Warnings**

- The garment is not a wound dressing - wound(s) should be fully covered by dressing(s).
- If any unexpected discomfort or wound irritation occurs, please consult your doctor or nurse.
- Do not use, if you have a known allergy or sensitivity to materials listed.
- Device is not indicated for wounds located on the chest or back.

**Potential side effects**

Dermatitis, infection and skin irritation or skin abrasion.

**Care instructions**


- Wash with similar colours in a washing bag on a delicate cycle.
- Use of chlorine based disinfection will decrease garment durability and function.

**SIZING CHART**

Centimetres	Small	Medium	Large	X-Large	XX-Large				
Chest/Bust (cm)	86	96	106	116	126	126	143		
Upper Arm (cm)	28 1/2	31	31	33 1/2	33 1/2	36	36	38 1/2	41
Inches	Small	Medium	Large	X-Large	XX-Large				
Chest/Bust (in)	34	38	42	42	46	46	50	50	55
Upper Arm (in)	11 1/4	12 1/4	12 1/4	13 1/4	13 1/4	14 1/4	14 1/4	15 1/4	16 1/4

Material composition:  
(85% Polyester, 15% Elastane)  
(Excludes Latex). Tricot Knit made  
of Polyurethane/Polyester Polyamide.

If an individual measurement falls between sizes, select smaller size. If Upper Arm and Chest sizes differ, select the smaller size.  
See below chart for additional guidance.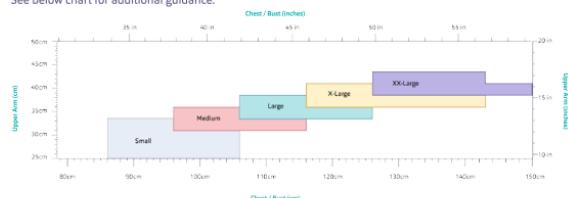

Patient Information Website  
[www.hidrawear.com](http://www.hidrawear.com)

Do not use if packaging  
is damaged

Dispose with  
appropriate care

Single Patient -  
multiple use

Manufactured by:  
HidraMed Solutions,  
Business Innovation Centre,  
NU1 Galway, H91 V207,  
Ireland

BS Partnership Ltd  
41 Dan-Y-Croes Road  
Cardiff CF23 6NB, UK

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