**INSTRUCTIONS FOR USE**
**Garment Application**

1. The care tag indicates the reverse and rear of the garment.
2. Briefs are worn underneath or in place of regular underwear.
3. Don the briefs as you would with standard boxer briefs.

**Dressing Application**

1. Insert the dressing(s) inside the garment and place in the desired area of the groin / buttocks region.
2. Use external fastener as directed in the dressing instructions for use, ensuring that you tuck in the tabs of your fastener into the garment holes.

**Dressing / Garment Removal**

1. Remove and discard the dressing as directed in the dressing instructions for use.
2. Remove the garment as done when removing regular briefs.

**Indications for use**

The garment is indicated for the retention of a dressing in the buttocks and groin.

**Intended user**

Intended for home use.

**Sizing**

Select your size as per sizing instructions.

**Guidelines**

1. Garment may be worn both throughout the day and/or overnight.
2. Check the garment for damage prior to use.

**Precautions**

1. Ensure garment is clean before use. Soiled garments should not be used.
2. Wash garment as required. See care instructions.
3. Change dressing as directed in the dressing instructions for use or as recommended by your doctor or nurse.

**Warnings**

1. The garment is not a wound dressing – wound(s) should be fully covered by dressing(s).
2. If any unexpected discomfort or wound irritation occurs, please consult your doctor or nurse.
3. Do not use if you have a known allergy or sensitivity to materials listed.
4. Device is not indicated for wounds located in areas other than the buttocks and groin region.

**Potential side effects**

Dermatitis, infection and skin irritation or skin abrasion.

**Care instructions**

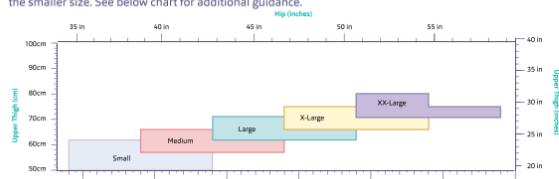
1. Wash with similar colours in a washing bag on a delicate cycle.
2. Use of chlorine based disinfection will decrease garment durability and function.

**SIZING CHART**

Centimetres	Small	Medium	Large	X-Large	XX-Large	
Hip (cm)	88	98	98 - 108	108 - 118	118 - 128	128 - 138
Upper Thigh (cm)	53	57	57 - 62	62 - 66	66 - 71	71 - 75
Inches	Small	Medium	Large	X-Large	XX-Large	
Hip (in)	35	39	39 - 43	43 - 46 1/2	46 1/2 - 50 1/2	50 1/2 - 54 1/2
Upper Thigh (in)	21	22 1/2	22 1/2 - 24 1/2	24 1/2 - 26	26 - 28	28 - 30

**Material composition:**  
78% Micro Modal, 22% Elastane  
(Exclusive of Trim). Trim composed of Polyester, Polyamide, Elastane, Polyurethane, Transparent Rubber Print

If an individual measurement falls between sizes, select smaller size. If Upper Thigh and Upper Hip sizes differ, select the smaller size. See below chart for additional guidance.

Patient Information Website  
[www.hidrawear.com](http://www.hidrawear.com)

Manufactured by:  
Business Innovation Centre,  
NUI Galway, Galway, H91 V207,  
Ireland

UK REP

BS Partnership Ltd  
41 Dan-Y-Coed Road  
Cardiff CF23 6NB, UK

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